IMPORTANT INFORMATION - PLEASE READ BEFORE USE
INFORMATIONS IMPORTANTES - À LIRE AVANT UTILISATION
INFORMAZIONI IMPORTANTI - LEGGERE PRIMA DELL'USO
WICHTIGE INFORMATIONEN - VOR GEBRAUCH BITTE LESEN
INFORMACIÓN IMPORTANTE - LÉALA ANTES DE UTILIZAR ESTE PRODUCTO
重要な情報 - 使用前にお読みください
ВАЖНА ИНФОРМАЦИЯ - МОЛЯ, ПРОЧЕТЕТЕ ПРЕДИ УПОТРЕБА
VAŽNE INFORMACIJE - PROČITATI PRIJE UPORABE
DŮLEŽITÉ INFORMACE - PŘEČTĚTE SI PŘED POUŽITÍM
VIGTIGE OPLYSNINGER - SKAL LÆSES INDEN BRUG
BELANGRIJKE INFORMATIE - VOOR GEBRUIK DOORLEZEN
OLULINE TEAVE - PALUN LUGEGE ENNE KASUTAMIST
TÄRKEITÄ TIETOJA - LUETTAVA ENNEN KÄYTTÖÄ
ΣΗΜΑΝΤΙΚΕΣ ΠΛΗΡΟΦΟΡΙΕΣ - ΠΑΡΑΚΑΛΟΥΜΕ ΔΙΑΒΑΣΤΕ ΠΡΙΝ ΑΠΟ ΤΗ ΧΡΗΣΗ
FONTOS TUDNIVALÓK - HASZNÁLAT ELŐTT OLVASSA EL
SVARĪGA INFORMĀCIJA - LŪDZU, IZLASIET PIRMS LIETOŠANAS
SVARBI INFORMACIJA - PERSKAITYKITE PRIEŠ NAUDODAMI
VIKTIG INFORMASJON - LES FØR BRUK
WAŻNE INFORMACJE - NALEŻY PRZECZYTAĆ PRZED UŻYCIEM
INFORMAÇÕES IMPORTANTES - LER ANTES DE UTILIZAR
INFORMAȚII IMPORTANTE - VĂ RUGĂM SĂ LE CITIȚI ÎNAINTEA UTILIZĂRII
DÔLEŽITÉ INFORMÁCIE - PREČÍTAJTE SI ICH PRED POUŽITÍM
POMEMBNE INFORMACIJE - PREBERITE PRED UPORABO
VIKTIG INFORMATION - LÄS FÖRE ANVÄNDNING
ÖNEMLİ BİLGİ - LÜTFEN KULLANMADAN ÖNCE OKUYUN

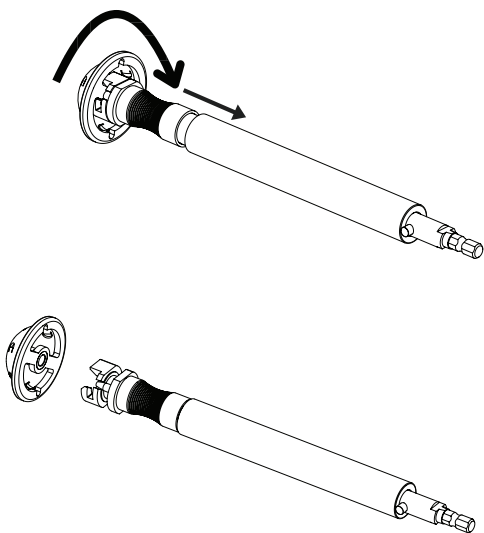
Contents (English - EN).....	1
Contenu (Français - FR).....	11
Contenuto (Italiano - IT).....	21
Inhalt (Deutsch - DE).....	31
Índice (Español - ES).....	42
内容（日本語 - JA）.....	52
Съдържание (Български - BG).....	62
Sadržaj (Hrvatski - HR).....	73
Obsah (Česky - CS).....	83
Indhold (Dansk - DA).....	93
Inhoud (Nederlands - NL).....	103
Sisukord (Eesti - ET).....	113
Sisältö (Suomi - FI).....	123
Περιεχόμενα (Ελληνικά - EL).....	133
Tartalom (Magyar - HU).....	144
Saturs (Latviski - LV).....	154
Turiny (Lietuvių - LT).....	164
Innhold (Norsk - NO).....	174
Spis treści (Polski - PL).....	184
Índice (Português - PT).....	194
Cuprins (Română - RO).....	204
Obsah (Slovenčina - SK).....	215
Vsebina (Slovenščina - SL).....	225
Innehåll (Svenska - SV).....	235
İçindekiler (Türkçe - TR).....	245

Humeral Socket Reamer/Planer Driver and Humeral Socket Reamer/Planer Driver Sleeve Assembly/Disassembly Instructions for Use and Cleaning and Reprocessing Instructions


The following instructions describe how to assemble and disassemble the Humeral Socket Reamer/Planer Driver and the Humeral Socket Reamer/Planer Driver Sleeve. The steps described below are for **DISASSEMBLY** of the device prior to cleaning and sterilization and should be reversed for assembly prior to use.


Note: Reference the reprocessing instructions below for the cleaning and sterilization instructions.

- Step 1** Remove the Humeral Reamer/Humeral Planer (Humeral Reamer shown below) from the Humeral Socket Reamer/Planer Driver by pulling the Thumb Trigger proximally and rotating the Humeral Reamer/Humeral Planer.

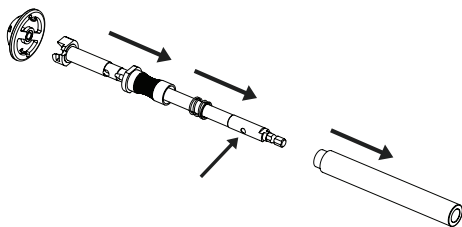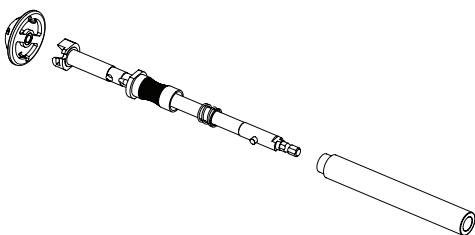- Step 2** Remove the Humeral Socket Reamer/Planer Driver Sleeve by depressing the Release Button and sliding the Humeral Socket Reamer/Planer Driver Sleeve proximally. The Spring and Thumb Trigger should remain on the Humeral Socket Reamer/Planer Driver shaft.

Assembly Prior to Use

Prior to use, ensure that the device is appropriately assembled.

1. The Spring must be present and functional.
2. The stepped end of the Humeral Socket Reamer/Planer Driver Sleeve must go on the Humeral Socket Reamer/Planer Driver first.
3. Depress the Release Button and slide the Humeral Socket Reamer/Planer Driver Sleeve over it.
4. Assess fit of the Humeral Socket Reamer/Planer Driver Sleeve.
 - a. There should not be a lot of excess space between the Humeral Socket Reamer/Planer Driver Sleeve and the Release Button.

These instructions are in accordance with ISO 17664 and AAMI ST81. They apply to:

Reusable surgical instruments and accessories supplied by Tecomet and intended for reprocessing in a health care facility setting. All Tecomet instruments and accessories may be safely and effectively reprocessed using the manual or combination manual/automated cleaning instructions and sterilization parameters provided in this document **UNLESS otherwise noted in instructions accompanying a specific instrument**.

In countries where reprocessing requirements are more stringent than those provided in this document it is the responsibility of the user/processor to comply with those prevailing laws and ordinances.

These reprocessing instructions have been validated as being capable of preparing reusable Tecomet instruments and accessories for surgical use. It is the responsibility of the user/hospital/health care provider to ensure that reprocessing is performed using the appropriate equipment, materials and that personnel have been adequately trained in order to achieve the desired result; this normally requires that equipment and processes are validated and routinely monitored. Any deviation by the user/hospital/health care provider from these instructions should be evaluated for effectiveness to avoid potential adverse consequences.

Materials & Restricted Substances

- For indication that the device contains a restricted substance or material of animal origin see product label.

Product Description

- See Intended Use

Intended Use

- The Humeral Socket Reamer/Planer Driver is intended for use during Total Shoulder Arthroplasty (TSA). The instrument provides a means for holding and driving a humeral reamer over a guide pin during preparation of the humerus to receive an implant in a reverse shoulder arthroplasty procedure. The device may be operated manually or by means of a powered surgical driver.

Intended Patient Population

- The device is prescriptive; therefore, a knowledgeable orthopedic surgeon may utilize the device on any patient he or she deems applicable.

Indications for Use

- The Humeral Socket Reamer/Planer Driver is indicated for use in Total Shoulder Arthroplasty (TSA) procedures to hold and drive a humeral reamer over a guide pin, facilitating the preparation of the humerus for implant placement.

Expected Clinical Benefits

- When used as intended, the Humeral Socket Reamer/Planer Driver aids in reaming and preparing the humerus for implant placement.

Combination of Medical Devices

- The Humeral Socket Reamer/Planer Driver connects to the Humeral Reamer/Planer. Reference the assembly/disassembly section.

Contra-indications

- The system or device is for prescription use and should only be used by qualified healthcare personnel. There are no contraindications for the system or device.

Intended User

- The system or device is prescription-only and therefore should be used by qualified orthopedic surgeons trained in the respective surgical technique.

Performance Characteristics

- The performance and safety of the system or device have been established, and it represents the current state of the art when used as intended.

Device Life

- The life expectancy of a reusable instrument and its accessories are dependent on the frequency of use and the care and maintenance the instruments receive. However, even with proper handling as well as correct care and maintenance, reusable instruments and accessories should not be expected to last indefinitely. Therefore, an accurate estimate of end of life is not possible for these types of manual, reusable devices.
- Instruments and accessories should be inspected for damage and wear before each use. Instruments and accessories that show signs of damage or excessive wear should not be used.

Disposal

At the end of the device's life safely dispose of the device in accordance with local procedures and guidelines.

Any device that has been contaminated with potentially infectious substances of human origin (such as bodily fluids) should be handled according to hospital protocol for infectious medical waste. Any disposable or defective device that contains sharp edges should be discarded according to hospital protocol in the appropriate sharps container.

Adverse Events & Complications

All surgical operations carry risk. The following are frequently encountered adverse events and complications related to having a surgical procedure in general:

- Delay to surgery caused by missing, damaged or worn instruments.
- Tissue injury and additional bone removal due to blunt, damaged or incorrectly positioned instruments.
- Infection and toxicity due to improper processing.

Adverse events to user:

Cuts, abrasions, contusions or other tissue injury caused by burs, sharp edges, impaction, vibration or jamming of instruments.

Adverse Events & Complications - Reporting of Serious Incidents

Serious Incident Reporting (EU)

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established. Serious incident means any incident that directly or indirectly led, might have led or might lead to any of the following:

- The death of a patient, user, or other person,
- The temporary or permanent serious deterioration of a patient's, user's, or other person's state of health,
- A serious public health threat.

Where further information is desired, please contact your local Tecomet sales representative. For instruments produced by another legal manufacturer, reference the manufacturer's instructions for use.

WARNINGS

- Tecomet reusable instruments and accessories that are provided **NON-STERILE**  must be cleaned and sterilized according to these instructions prior to each use.
- Personal Protective Equipment (PPE) should be worn when handling or working with contaminated or potentially contaminated instruments and accessories.
- Caution should be exercised while handling, cleaning, or wiping instruments and accessories with sharp cutting edges, tips, and teeth.
- Ethylene oxide (EO), gas plasma and dry heat sterilization methods are **not recommended** for sterilization of Tecomet reusable instruments. Steam (moist heat) is the recommended method.
- Saline and cleaning/disinfection agents containing aldehyde, chloride, active chlorine, bromine, bromide, iodine or iodide are corrosive and **should not** be used.
- **Do not allow biologic soil to dry on contaminated devices.** All subsequent cleaning and sterilization steps are facilitated by not allowing blood, body fluids and tissue debris to dry on used instruments.
- Automated cleaning using a washer/disinfector alone **may not** be effective for instruments with lumens, blind holes, cannulas, mated surfaces and other complex features. A thorough manual cleaning of such device features is recommended before any automated cleaning process.
- Metal brushes and scouring pads must not be used during manual cleaning. These materials will damage the surface and finish of the instruments. Use only soft bristle nylon brushes with different shapes, lengths and sizes to aid with manual cleaning.
- When processing instruments do not place heavy devices on top of delicate instruments.
- **Use of hard water should be avoided.** Softened tap water may be used for most rinsing however purified water should be used for final rinsing to prevent mineral deposits.
- Do not process instruments with polymer components at temperatures equal to or greater than 140°C/285°F because severe surface damage to the polymer will occur.
- Oils or silicone lubricants **should not** be used on surgical instruments.

Limitations on Reprocessing

- Repeated processing according to these instructions has minimal effect upon metal Tecomet reusable instruments and accessories unless otherwise noted. End of life for stainless steel or other

metal surgical instruments is generally determined by wear and damage incurred during the intended surgical use.

- Tecomet instruments comprised of polymers or incorporating polymer components can be sterilized using steam. However, they are not as durable as their metal counterparts. If polymer surfaces show signs of excessive surface damage (e.g. crazing, cracks or delamination), distortion or are visibly warped they should be replaced. Contact your Tecomet representative for your replacement needs.
- Non-foaming, neutral pH enzymatic and cleaning agents are recommended for processing Tecomet reusable instruments and accessories.
- Alkaline agents with a pH of 12 or less may be used to clean stainless steel and polymer instruments in countries where required by law or local ordinance; or where prion diseases such as Transmissible Spongiform Encephalopathy (TSE) and Creutzfeld-Jakob Disease (CJD) are a concern. **It is critical that alkaline cleaning agents are completely and thoroughly neutralized and rinsed from the devices or degradation may occur that limits the device life.**

REPROCESSING INSTRUCTIONS

Point of Use

- Remove excess biologic soil from the instruments with a disposable wipe. Place devices in a container of distilled water or cover with damp towels.

Note: Soaking in proteolytic enzyme solution prepared according to the manufacturer will facilitate cleaning especially in instruments with complex features such as lumens, mating surfaces, blind holes and cannulas.

- If instruments cannot be soaked or maintained damp then they should be cleaned as soon as possible after use to minimize the potential for drying prior to cleaning.

Containment and Transportation

- Used instruments must be transported to the decontamination area for reprocessing in closed or covered containers to prevent unnecessary contamination risk.

Preparation for Cleaning

- These instruments are designed to come apart and must be disassembled prior to cleaning. Reference the Disassembly instructions above for more detail.

Note: All recommended disassembly will be possible by hand. Never use tools to disassemble instruments beyond what is recommended.

- All cleaning solutions should be prepared at the dilution and temperature recommended by the manufacturer. Softened tap water may be used to prepare cleaning solutions.

Note: Fresh cleaning solutions should be prepared when existing solutions become grossly contaminated (turbid).

Manual Cleaning Steps

- **Step 1:** Prepare a proteolytic enzyme solution according to the manufacturer's instructions.
- **Step 2:** Completely submerge instruments in the enzyme solution and gently shake them to remove trapped bubbles. Actuate instruments with hinges or moving parts to ensure contact of the solution with all surfaces. Lumens, blind holes and cannulations should be flushed with a syringe to remove bubbles and ensure contact of the solution with all instrument surfaces.
- **Step 3:** Soak instruments for a minimum of 10 minutes. While soaking, scrub surfaces using a soft nylon-bristled brush until all visible soil has been removed. Actuate moveable mechanisms. Particular attention should be given to crevices, hinged joints, box locks, instrument teeth, rough surfaces and areas with moving components or springs. Lumens, blind holes and cannulas should

be cleaned using a snug fitting round nylon bristle brush. Insert the snug fitting round brush into the lumen, blind hole or cannula with a twisting motion while pushing in and out multiple times.

Note: All scrubbing should be performed below the surface of the enzyme solution to minimize the potential of aerosolizing contaminated solution.

- **Step 4:** Remove the instruments from the enzyme solution and rinse in tap water for a minimum of one (1) minute. Actuate all moveable and hinged parts while rinsing. Thoroughly and aggressively flush lumens, holes, cannulas and other difficult to access areas.
- **Step 5:** Prepare an ultrasonic cleaning bath with detergent and de-gas according to the manufacturer's recommendations. Completely submerge instruments in the cleaning solution and gently shake them to remove any trapped bubbles. Lumens, blind holes and cannulations should be flushed with a syringe to remove bubbles and ensure contact of the solution with all instrument surfaces. Sonically clean the instruments at the time, temperature and frequency recommended by the equipment manufacturer and optimal for the detergent used. A minimum of ten (10) minutes is recommended.

Notes:

- **Separate stainless steel instruments from other metal instruments during ultrasonic cleaning to avoid electrolysis.**
- **Fully open hinged instruments.**
- **Use wire mesh baskets or trays designed for ultrasonic cleaners.**
- **Regular monitoring of sonic cleaning performance by means of an ultrasonic activity detector, aluminum foil test, TOSI™ or SonoCheck™ is recommended.**
- **Step 6:** Remove the instruments from the ultrasonic bath and rinse in purified water for a minimum of one (1) minute or until there is no sign of residue detergent or biologic soil. Actuate all moveable and hinged parts while rinsing. Thoroughly and aggressively flush lumens, holes, cannulas and other difficult to access areas.
- **Step 7:** Dry instruments with a clean, absorbent non-shedding lint free cloth. Clean, filtered compressed air may be used to remove moisture from lumens, holes, cannulas and difficult to access areas.

Combination Manual/Automated Cleaning Steps

- **Step 1:** Prepare a proteolytic enzyme solution according to the manufacturer's instructions.
- **Step 2:** Completely submerge instruments in the enzyme solution and gently shake them to remove trapped bubbles. Actuate instruments with hinges or moving parts to ensure contact of the solution with all surfaces. Lumens, blind holes and cannulations should be flushed with a syringe to remove bubbles and ensure contact of the solution with all instrument surfaces.
- **Step 3:** Soak instruments for a minimum of 10 minutes. Scrub surfaces using a soft nylon-bristled brush until all visible soil has been removed. Actuate moveable mechanisms. Particular attention should be given to crevices, hinged joints, box locks, instrument teeth, roughened surfaces, and areas with moving components or springs. Lumens, blind holes and cannulas should be cleaned using a snug fitting round nylon bristle brush. Insert the snug fitting round nylon bristle brush into the lumen, blind hole or cannula with a twisting motion while pushing in and out multiple times.

Note: All scrubbing should be performed below the surface of the enzyme solution to minimize the potential of aerosolizing contaminated solution.

- **Step 4:** Remove the instruments from the enzyme solution and rinse in tap water for a minimum of one (1) minute. Actuate

all moveable and hinged parts while rinsing. Thoroughly and aggressively flush lumens, holes, cannulas and other difficult to access areas.

- **Step 5:** Place instruments in a suitable validated washer/disinfector. Follow the washer/disinfector manufacturer's instructions for loading the instruments for maximum cleaning exposure; e.g. open all instruments, place concave instruments on their side or upside down, use baskets and trays designed for washers, place heavier instruments on the bottom of trays and baskets. If the washer/disinfector is equipped with special racks (e.g. for cannulated instruments) use them according to the manufacturer's instructions.
- **Step 6:** Process instruments using a standard washer/disinfector instrument cycle according to the manufacturer's instructions. The following minimum wash cycle parameters are recommended:

Cycle	Description
1	Pre-wash • Cold Softened Tap Water • 2 minutes
2	Enzyme Spray & Soak • Hot Softened Tap Water • 1 minute
3	Rinse • Cold Softened Tap Water
4	Detergent Wash • Hot Tap Water (64-66°C/146-150°F) • 2 minutes
5	Rinse • Hot Purified Water (64-66°C/146-150°F) • 1 minute
6	Hot Air Dry (116°C/240°F) • 7 – 30 minutes

Notes:

- The washer/disinfector manufacturer's instructions should be followed.
- A washer/disinfector with demonstrated efficacy (e.g. FDA approval, validated to ISO 15883) should be used.
- Dry time is shown as a range because it is dependent upon the load size placed into the washer/disinfector.
- Many manufacturers pre-program their washer/disinfectors with standard cycles and they may include a thermal low-level disinfection rinse after the detergent wash. The thermal disinfection cycle should be performed to achieve a minimum value $A_0 = 600$ (e.g. 90°C/194°F for 1 minute according to ISO 15883-1) and is compatible with Tecomet instruments.
- If a lubrication cycle is available that applies to water-soluble lubricant such as Preserve®, Instrument Milk or equivalent material intended for medical device application it is acceptable to use on Tecomet Instruments unless otherwise indicated.

Disinfection

- Tecomet instruments must be terminally sterilized prior to use. See sterilization instructions below.
- Low level disinfection may be used as part of a washer/disinfector cycle but the devices must also be sterilized before use.

Drying

- Dry instruments with a clean, absorbent non-shedding lint free cloth. Clean, filtered compressed air may be used to remove moisture from lumens, holes, cannulas and difficult to access areas.

Inspection & Testing

- After cleaning, all devices should be thoroughly inspected for residue biologic soil or detergent. If contamination is still present repeat the cleaning process.

- Visually inspect each device for completeness, damage and excessive wear. If damage or wear is observed that might compromise the function of the device, do not process them further and contact your Tecomet representative for a replacement.
- When inspecting devices look for the following:
 - Movable parts should operate smoothly throughout the intended range of motion.
 - Where instruments form part of a larger assembly, check that all components are available and assemble readily.

Maintenance and Lubrication

- After cleaning and before sterilization, instruments with moving parts (e.g. hinges, box-locks, sliding or rotating parts) should be lubricated with a water-soluble lubricant such as Preserve®, Instrument Milk or equivalent material intended for medical device application. Always follow the lubricant manufacturer's instructions for dilution, shelf life and application method.

Packaging for Sterilization

- Single devices may be packaged in an approved (e.g. FDA cleared or ISO 11607 compliant) medical grade sterilization pouch or wrap. Care should be used when packaging so that the pouch or wrap is not torn. Devices should be wrapped using the double wrap or equivalent method (ref: AAMI ST79, AORN Guidelines).
- Reusable wraps are not recommended.
- Instruments may be packaged in an approved (e.g. FDA cleared or ISO 11607 compliant) general-use perforated tray or case along with other devices under the following conditions:
 - Arrange all devices to allow access of steam to all surfaces. Open hinged devices and ensure devices are disassembled if it is recommended.
 - The case or tray must be wrapped in an approved (e.g. FDA cleared or ISO 11607 compliant) medical grade sterilization wrap by following the double wrap method or equivalent (ref: AAMI ST79, AORN Guidelines).
 - Follow the case/tray manufacturer's recommendations for loading and weight. Total weight of the wrapped case or tray should not exceed 11.4kg/25lbs.
- Instruments may be packaged in an approved (e.g. FDA cleared or ISO 11607 compliant) rigid container system (i.e. those with filters or valves) along with other devices under the following conditions:
 - The container manufacturer's recommendations should be followed regarding preparation, maintenance and use of the container.
 - Arrange all devices to allow access of steam to all surfaces. Open hinged devices and ensure devices are disassembled if recommended.
 - Follow the container manufacturer's recommendations for loading and weight. Total weight of a filled container system should not exceed 11.4kg/25lbs.

Sterilization

- Moist heat/steam sterilization is the recommended method for Tecomet instruments.
- Use of an approved chemical indicator (class 5) or chemical emulator (class 6) within each sterilization load is recommended.
- Always consult and follow the sterilizer manufacturer instructions for load configuration and equipment operation. Sterilizing equipment should have demonstrated efficacy (e.g. FDA clearance, EN 13060 or EN 285 compliance,). Additionally the manufacturer's recommendations for installation, validation, and maintenance should be followed.
- Validated exposure times and temperatures to achieve a 10^{-6} sterility assurance level (SAL) are listed in the following table.

Cycle Type	Temperature	Exposure Time
United States Recommended Parameters		
Pre-vacuum / Vacuum Pulse	132°C/270°F	4 minutes
European Recommended Parameters		
Pre-vacuum / Vacuum Pulse	134°C/273°F	3 minutes

Drying & Cooling

- The recommended drying time for single wrapped instruments is 30 minutes unless otherwise noted in device specification instructions.
- Drying times for instruments processed in containers and wrapped trays can vary depending upon the type of packaging, type of instruments, type of sterilizer and total load. A minimum dry time of 30 minutes is recommended, but to avoid wet packs, extended dry times greater than 30 minutes may be needed for larger loads under certain conditions or if otherwise recommended in accompanying documentation. For large loads verification of dry times by the health care provider is recommended.
- **A 30 minute minimum cooling time is recommended after drying but longer times may be necessary because of load configuration, ambient temperature and humidity, device design and packaging used.**

Note: Disinfection/steam sterilization parameters recommended by the World Health Organization (WHO) for reprocessing instruments where there is a concern about TSE/CJD contamination are: 134°C/273°F for 18 minutes. Tecomet devices are compatible with these parameters.

Storage

- Sterile packaged instruments should be stored in a designated, limited access area that is well ventilated and provides protection from dust, moisture, insects, vermin and temperature/humidity extremes.

Note: Inspect every package before use to ensure that the sterile barrier (e.g. wrap, pouch or filter) is not torn, perforated, shows signs of moisture or appears to be tampered with. If any of those conditions are present then the contents are considered non-sterile and should be re-processed through cleaning, packaging and sterilization.

Symbols Used on Labeling:

	Caution
	Non-Sterile
	U.S Federal law restricts this device to sale by or on the order of a physician
	CE Mark with Notified Body #
	Authorized Representative in the European Community
	Manufacturer
	Date of Manufacture
	Lot Number
	Catalog Number
	Consult Instructions for Use
	Medical Device
	Packaging Unit
	Country of Manufacture
	Distributor
	Swiss Authorized Representative
	Importer
	Unique Device Identifier

Additional Labeling:

“REMOVE FOR CLEANING & STERILIZATION”	Device should be disassembled prior to cleaning and sterilization.
“CLEANING PORT”	Cleaning port on the Humeral Socket Reamer/Planer Driver
“ACTUATE TO CLEAN AND EXPOSE PORT”	Instructions to move the Collar on the Humeral Socket Reamer/Planer Driver

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Symmetry Medical Manufacturing, Inc.
Symmetry Medical Polyvac S.A.S